

# Start Here

1

## Start Up


### MediaMed Kiosk



To turn on the touch-screen kiosk, locate the **POWER BUTTON** on the bottom-front of the screen. Press the button lightly once to turn the kiosk on. **DO NOT HOLD THE POWER BUTTON.** Allow a few minutes to boot up to the MediaMed main menu. **DO NOT** turn off daily.

If the system does not boot up to the MediaMed main menu, press the **MEDIAMED** icon in the middle of the screen.

### Camera

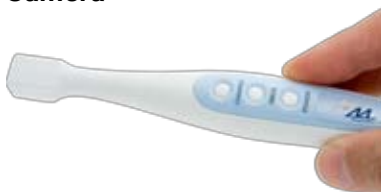

Plug the camera into the **USB** port on right side of kiosk. The camera will turn on automatically with kiosk.

2

## Navigating the MediaMed Interface

### Main Menu

Initial selections are made from this screen.



### Setting up a new patient

Press **NEW PATIENT** button.

### YOU MUST COMPLETE ALL OF THE FOLLOWING STEPS TO CONTINUE.

- Using the on-screen keyboard, type in the system password: **MEDIAMED**
- Define patient using the keypad on touch screen. You may use a chart number, first initial and last name, or a combination of the two. For example "J. Smith 1234 or 24-4567." **DO NOT PUT THE PATIENT'S FULL NAME** together with **FULL** Social Security Number.
- Select **GENDER** (Male/Female)
- Enter patient's **YEAR OF BIRTH**

Patient information, including pictures taken and videos watched, will be saved under this defined patient name.

You are now able to make your next selection.

### Language



The system will automatically play in English. Press the desired **LANGUAGE BUTTON**,

at bottom of screen, to choose a playback language. The selected language will be indicated in red.

You must choose a language before making any selection. The language will not change if either a video or the screen saver are already in progress.

### Patient Education

**IMPORTANT** to view patient education videos, you will be prompted to create a new patient. Please see the above instructions on setting up a new patient.

Press the **PATIENT EDUCATION** button

The videos are separated into sections as follows:

#### EXAMINATION

- Adult First Visit
- Pediatric First Visit

#### PERIODONTICS

- Adult Prophy
- Scaling & Root Planing
- Implants
- Crown Lengthening
- Arestin
- Bone Graft

#### ENDODONTICS

- Root Canal Therapy
- Apicoectomy
- Amalgam Plug
- Root Amputation
- Hemisection

#### CROWN & BRIDGE

- Dental Bridges
- Dental Crowns
- Crown Material Comparison
- Onlay
- Veneer

#### ORTHODONTICS

- Invisalign
- Night Guard

#### COSMETIC

- Home Bleaching
- Office Bleaching
- Veneer
- Invisalign

#### RESTORATIVE

- Amalgam Fillings
- Composite Fillings
- Inlay
- Fillings Material Comparison

#### EXTRACTION

- Tooth Extraction
- Wisdom Teeth
- Bone Graft
- Post-Op Instructions

#### REMOVABLE PROSTHODONTICS

- Complete Denture
- Partial Denture
- Reline
- Stayplate
- Immediate Denture
- Post-Op Instructions

#### OHI

- Oral Hygiene Instructions

Please note that some of the above videos may be not be available in the current system software release.

3

## Patient Education Menu

You can select as many videos as desired.



A yellow dot will appear in the button indicating that the

video has been selected.

To deselect a video, press the button a second time.

Press **PLAY VIDEOS** when all have been selected. The videos will play in the order that you selected them.

Press the **BACK** button to return to the Main Menu

While videos are playing:



- **BACK** re-starts video from the beginning
- **PAUSE** to stop video
- **MAIN MENU** to exit



When the patient is finished viewing your selected videos, you can select more educational videos, select the camera feature, or press **RESET USER** for a new patient.

## 4 INTRA-ORAL CAMERA


Press the **CAMERA** button to activate the Intra-Oral Camera

There are three buttons on the camera:

- **POWER** turns light on and off.
- **ZOOM** use to zoom in and out.
- **FREEZE** takes a picture.

### The Camera Menu

- **TAKE PHOTO** press button or **FREEZE** button on the camera to capture an image.

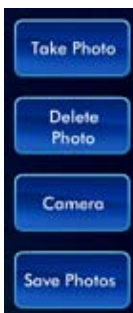

You can capture up to 5 images at a time.

- **DELETE** press button to delete. Will delete in order from last picture taken.
- **CAMERA** enter stand-by mode and prepare to take next picture.
- **SAVE PHOTOS** will save to the server. Please allow 10-15 seconds for processing.

## 5 Entertainment


Insert either an audio CD or DVD movie into the drive slot, located on the right side of kiosk. Select the **MUSIC** or **DVD** button from the Main Menu.



If no CD or DVD is inserted in the machine, you will get an alert. Please insert a CD or DVD and try again. Allow 15 seconds for the disc to load.

### DVD Movies

During DVD playback, you can pause or resume playback using the button in the upper right corner. To exit DVD playback, press **MAIN MENU** on the lower left.

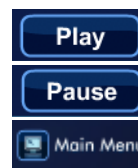

### Music CD

The educational tips will play while the music plays. Use the buttons in the upper right corner to control playback. To exit hit **MAIN MENU** on the lower left.



## 6 Educational Tips



There are two ways to activate the Educational Tips.

- From the Main Menu, press **EDUCATIONAL TIPS**.
- After five minutes of system inactivity the Educational Tips screen-saver will automatically begin playback.



To exit the Educational Tips module, press **MAIN MENU** at the lower left. If the system is in screen saver mode, simply tap anywhere on the screen to return to the main menu.

Tips will play in the currently selected language. Press a language button to change the playback language.

## 7 Additional Support

Additional support is available on our web site, **WWW.MEDIAMED.COM**, or by contacting our technical support department by telephone, at 1-(866)-354-3244.



MediaMed Inc.  
236 N. Santa Cruz Avenue, #223  
Los Gatos, CA 95030  
866.354.3244  
www.MediaMed.com